



SAFE life™


Fentanyl (FTY) Liquid & Powder Drug Test Strip

- Fentanyl Harm Reduction
- Rapid Results
- Accurate

SAFElife™ Fentanyl (FTY) Liquid & Powder Drug Test Strip is an immunochromatographic assay for the qualitative determination of Fentanyl in liquid, powder or pill samples.

Background

Fentanyl (FTY) is a potent, synthetic opioid analgesic with a rapid onset and short duration of action. Fentanyl is approximately 50 times stronger than heroin and 100 times stronger than morphine. It has become a major contributor to fatal and nonfatal overdoses in the U.S. While it is used clinically for severe pain, most fentanyl-related overdose cases are linked to its presence in illegal drug markets. In response to this crisis, the **SAFElife™ Fentanyl (FTY) Liquid & Powder Drug Test Strip can determine if fentanyl is present in a liquid or powder, providing vital information to help with harm reduction.**

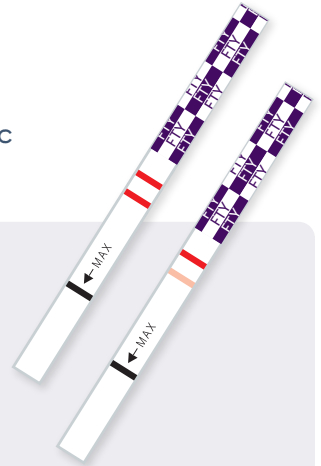

Features and Benefits

- Detects fentanyl and multiple fentanyl analogs
- No false positives with MDMA, Methamphetamine, Cocaine or Morphine
- Results at 5 minutes
- 99% Accurate

Drug (Identifier)	Fentanyl (FTY)
Calibrator	Fentanyl
Cut-off Level	10 ng/mL
Sample Types	Liquid or Powder

Test Procedure

Step 1 Add water

Powder (10mg)

Mix well

Water (5mL)

Step 2 Test

15 Seconds

MAX Marker Line

Step 3 Wait

5mins

Step 4 Result

Negative Preliminary positive Invalid

Item Code	Description	Box Size / Case Size	Complexity
WSPFTY-101	1-Strip; Drug Liquid / Powder Detection; Fentanyl 10 ng/mL	25/120	Forensic Use Only Harm Reduction Use

Rel: 2024/05/10